

Phonak Roger Portable Room Systems – Charging Routine

It is important that you charge the equipment contained in the Portable Room System before use, ideally overnight or a couple of hours before needed, particularly if you will be using the equipment all day. You can of course top up the charge half way through the day at lunchtime if needed.

A charging rack can charge up to 4 devices simultaneously. You plug the charging rack into the mains and then slot your devices into the charging racks.







Touchscreen Mic, Pass Around Mic and MultiMedia Hub slot straight into the charging rack and its micro-USB charging sockets. MultiMedia Hubs are often attached to a TV. If so it would tend to be semi permanently plugged into the mains socket at the back of the TV.

Roger Neckloop (and Roger On if you have one) need to plug in the black inserts that convert micro-USB to USB-C charging points.

Roger Table Mics (If part of the package) have their own charging plug, and you can charge both Table Mics together from the same charging cable with the provided double adapter.

The Roger DigiMaster speaker does not have an internal battery and needs to be mains powered in use. DigiMaster typically is placed in the corner of the room behind the lead person in the meeting / event pointing out to the room. It needs to be near to a wall socket, powered and ready to connect to the rest of the Portable Room System and ready to broadcast audio to all in the room.

DigiMaster tends to stay in one main room, the rest of the equipment is ultra portable with its custom case and easily moved from room to room around the building. **Any queries please just ask!**

			
Roger Touchscreen Mic	Pass-Around Mic	MultiMedia Hub	Roger Neckloop
			
DigiMaster Speaker	Charging Rack + Inserts	Phonak Roger On	Roger Table Mic x2