

Phonak Roger Portable Room Systems – How They Work



Phonak Roger Room Systems are a clever blend of high quality wireless remote microphone technology with the aim of aiding audibility in difficult to hear group and room situations.

In brief, listed are all the possible component parts you might find in a typical Phonak Roger Room System.

It's possible for us to supply part of and not all the following individual components as a bundle, but for the purpose of covering all bases, the following information will cover everything you need to know about all the permutations available.

A Phonak Roger Room System helps to overcome the problems associated with hearing at distance, and where reverberation and background noise is present.

It helps with audio devices and media, and generally aids inclusivity in difficult listening environments.

A Phonak Roger Room System consists of a 'Roger Touchscreen Mic' primary transmitter worn by the main person or speaker, one or more 'Pass-Around Mics' used by other people in the room and a MultiMedia Hub, which can be used to broadcast high quality audio from a TV or other audio device.

These transmitters send high quality audio wirelessly to a 'DigiMaster' speaker system, a high quality smart speaker that is enhanced for people with hearing loss and which broadcasts to everyone in the room.

The transmitted audio is also sent direct to Roger Neckloop. Neckloops are worn by people in the room with hearing loss that are wearing hearing aids with a telecoil (or loop program), active and selected.

This adds an extra layer of quality and speech intelligibility for people that might otherwise struggle hearing with the DigiMaster speaker alone.

If no hearing aids or loop program is available, another option is using earphones in conjunction with a Roger Neckloop. Volume adjustments are possible via the control down the side of the Neckloop.

We have an optional add-on 'Meeting Room Pack' for meeting room scenarios.

Consisting of a 'Roger On' primary wireless smart microphone transmitter and 'Roger Table Mic' transmitters, they work together, sending high quality audio wirelessly straight to the Roger Neckloop receivers. The meeting pack is a discreet design and is standalone.

Roger On is used for smaller, more fluid meetings and Table Mics are for bigger meetings round larger tables.

We do hope you found this information useful, any queries just ask.

FM Hearing Systems - 01535 279285 option 1 - help@fmhearingsystems.co.uk

Phonak Roger Portable Room Systems – How They Work

Main Parts of a Phonak Room System



Roger Touchscreen Mic



Roger Pass-Around Mic



Roger MultiMedia Hub



Phonak Roger Neckloop



DigiMaster Speaker



Charging Rack with Inserts



Phonak Roger On



Roger Table Mic x2 & Remote



Custom Case